Please read carefully and keep this package insert for future reference.

BAUSCH + LOMB

EasySept HYDRO+

Peroxide Solution

Cleans, Conditions, Removes Protein, Disinfects, Stores For All Soft Contact Lenses, Including Silicone Hydrogel Lenses, and Rigid Gas Permeable Lenses

WARNING:

Do not use flat lens case. Always use the enclosed special lens case and neutralising disc with EasySept Hydro+.

NEVER insert EasySept Hydro+ solution directly into the eye. DO NOT place your lens into the eye until complete neutralisation of the solution has occurred (minimum 4 hours).

Do not rinse lenses with EasySept Hydro+ solution prior to inserting lenses into your eyes. If you want to rinse your lenses, use a sterile saline solution.

Do not reuse or "top off" old solution left in your lens case. "Topping off" is the addition of fresh solution to the used solution that has been sitting in your lens case.

Directions for Soft Contact Lenses in black, for Rigid Gas Permeable Lenses in blue (also see Important Safety Information):

1. Always wash, rinse, and thoroughly dry your hands before handling your lenses.

2. Check that the special lens case includes the neutralising disc.

3. Stand the case on its lid on a flat surface. Open lens baskets. Place your left and right lenses on the domes inside the appropriate baskets (left side is green and marked with the letter "L"). Close the baskets taking care not to trap the lenses.

After both lenses are removed from the eyes, take each lens from its holder and place into the palm of your hand. Apply 2-4 drops of EasySept Hydro+ solution and gently rub lenses for 5 seconds on each side.

Place your left and right lenses on the domes inside the appropriate baskets (left and right). Close the baskets taking care not to trap the lenses. Some RGP users may experience a mild, temporary skin discolouration (bleaching) to their fingers or hands while rubbing their lenses. Always wash and rinse your hands after rubbing your lenses with EasySept Hydro+.

4. Hold the lens case lid and rinse each lens for 5 seconds in the baskets with EasySept Hydro+ solution over a sink/basin.

5. Fill the lens case up to the fill line with fresh EasySept Hydro+ solution then immediately place the holder basket into the lens case. Screw the lid on the case and store lenses in upright case for at least 4 hours. Do not shake or invert lens case once filled.

6. Your lenses will be ready to wear after soaking for 4 hours.

7. Once lenses are removed from the baskets, discard any remaining solution from the lens case, rinse the case with EasySept Hydro+ solution over a sink/basin, and leave the lid off the case to air dry.

8. If desired, each side of the lens may be rinsed for 5 seconds (for a total of 10 seconds) with a sterile saline prior to inserting.

Lenses may be stored in the unopened case until ready to wear, up to a maximum of 7 days. If stored longer than 7 days between wears you need to follow the cleaning and disinfecting steps outlined in the directions for use prior to lens insertion.

If not wearing contact lenses immediately, store in unopened lens case. Do not store your lenses in just normal saline, as saline will not disinfect.

Contents: 3% hydrogen peroxide with a phosphonic acid stabiliser, potassium chloride, propylene glycol, carbamide, poloxamer 181, citrate and phosphate buffers.

Actions: When EasySept Hydro+ is added to the special lens case, the neutralising disc causes a distinctive fizzing action, effectively cleaning, disinfecting, and removing protein from all soft and rigid gas permeable contact lenses. After four (4) hours the peroxide solution is completely neutralised into a preservative-free saline solution and your lenses are ready to use!

Indications (Uses): EasySept Hydro+ is indicated for the daily cleaning, removal of protein deposits, disinfection, and storage of soft (hydrophilic) contact lenses, including silicone hydrogel contact lenses, and rigid gas permeable contact lenses, as recommended by your eye care practitioner.

Contraindications (Reasons Not To Use):

If you are allergic to any ingredient in this product, do not use.

Failure to discard solution from lens case after each use or use of water to care for your lenses may lead to contamination resulting in eye infection, eye injury, and potential loss of vision. See accompanying instructions for additional Important Safety Information.

Do not reuse or "top off" old solution left in your lens case. "Topping off" is the addition of fresh solution to the used solution that has been sitting in your lens case.

Important Safety Information:

• Do not insert EasySept Hydro+ solution, or the lens, directly into the eye until complete neutralisation of the solution has occurred (minimum 4 hours).

• EasySept Hydro+ **IS NOT** a multi-purpose solution.

• Do not rinse lenses with EasySept Hydro+ solution prior to inserting lenses into your eyes. If you want to rinse lenses, use a sterile saline solution.

• Always follow the product directions for use. Failure to follow product directions may lead to vision loss.

• Always wash and dry hands before handling lenses.

• Do not use tap water, bottled water, or saliva with lenses or lens case.

• Only use fresh solution to clean and disinfect contact lenses.

• Always replace your solutions, lenses, and lens case as directed.

• To avoid contamination, do not touch tip of container to any surface. Replace cap after using.

• Not for use with heat (thermal) disinfection.

• Visit your eye care practitioner regularly.

Precautions:

• Always discard solution from lens case after each use.

• Keep the bottle tightly closed when not in use.

• Use before the expiry date marked on the carton and bottle.

• Discard bottle and lens case 90 days after opening or after 35 uses, whichever occurs first.

• Store at room temperature. Do not freeze.

• Do not use if tamper evident seal is missing or broken.

• Only use solution with specially designed EasySept Hydro+ lens case.

• Do not invert or shake lens case as residual unneutralised peroxide could remain on the cap.

KEEP OUT OF REACH OF CHILDREN.

Do not mix or substitute other lens care hydrogen peroxide products or lens cases. Do not use any other disinfection solutions in the lens case.

DO NOT USE OVER-THE-COUNTER GENERIC HYDROGEN PEROXIDE.

How Supplied: EasySept Hydro+ is available for purchase in sterile, 120 and 360 mL plastic bottles and supplied with a new peroxide lens case. Bottles and cartons are marked with a lot number and expiry date.

*TM are trademarks of Bausch & Lomb Incorporated or its affiliates.
©Bausch & Lomb Incorporated

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